


# MediGo Flex

Progressive Omni Solutions



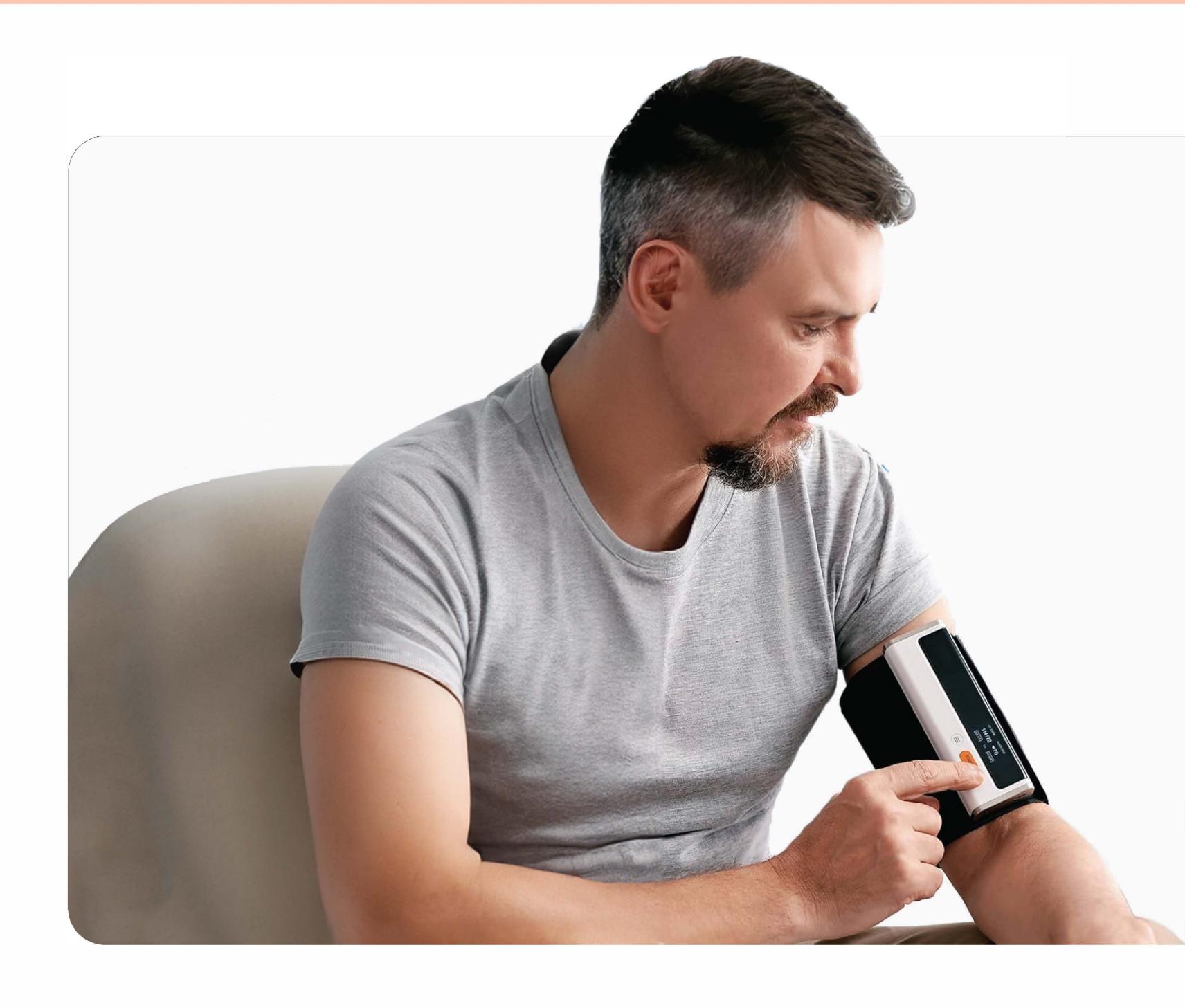

## CardioCheck

Wi-Fi Smart Blood
Pressure Monitor With ECG

Price:

₹14,999.00 /- inclusive all taxes

#### Features:



#### **Blood Pressure Monitoring**

Provides Accurate Blood Pressure, Pulse Rate Readings and ECG



#### App Integration

The Medigo Flex App Stores Your Data.



#### **ECG** Function

Integrated ECG Electrodes Detect Atrial Fibrillation, Tachycardia & Bradycardia.

#### Certifications:



FDA- Cleared Product




**CE - Certified** Product



Compliance with ISO

Colo Wi-Fi Smart Blood With ECG Pressure Monitor With ECG

#### In the Box:

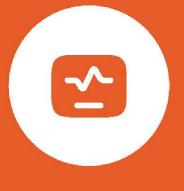

CardioCheck Duo Device



Charging cable



Quick guide & warranty card



User Manual



Rechargeable battery


MediGo Flex

## Product Specifications:


Name	Details
Model Number	MGF-BPMECG-01
Dimensions	135mm(L) x 45mm(W) x 20mm(H)
Weight	240g
Cuff Size	Adult cuff: 22-42cm
Display	LED for diastolic & heart rate readings
Blood Pressure Measurement	
Method	Oscillometric with automatic inflation and deflation
Range	0-300 mmHg
Accuracy	±3 mmHg
Pulse Rate Measurement	
Range	40-200 beats per minute
Accuracy	±2 beats per minute
ECG Function	
Type	Integrated ECG electrodes
Storage Condition	

Name	Details
Duration	Up to 30 seconds
Detection	Atrial Fibrillation, tachycardia, bradycardia
User Profiles	Up to 10 users with independent memory
Connectivity	
Bluetooth	4.0 for wireless data transmission
Wi-Fi	2412-2484 MHz
Compatible Apps	MediGo Flex Health Mate (iOS and Android)
Battery	
Type	Rechargeable Lithium-polymer
Life	Up to 1 month on a single charge
Charging Time	2 hours
Operating Conditions	
Temperature	25°C to 70°C
Humidity	10% to 95%, non-condensing
Warranty	1 Year

### Additional Features:

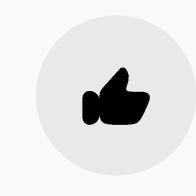



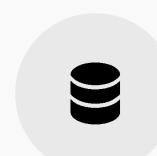







Instant Feedback

Rechargeable battery

Data Storage Multi-User Capability Apple Health Integration


Routine Reminders

## Steps to Use Blood pressure on CardioCheck Duo

- Power On Press the start button to turn on the device.
- Wrap the Cuff Securely place the cuff around your upper arm. ensure a snug fit but not too tight.
- Connect to App Open the medigo flex app and pair the device via bluetooth or wi-fi.
- Start Measurement Press the start button to begin taking your blood pressure and ECG reading.
- Stay Still Remain still and relaxed as the device records your readings.
- View Results Check your results on the device screen or in the mobile app.
- Save & Track Store your data in the app for long-term health tracking and analysis.

## Steps to Use the ECG Function on CardioCheck Duo


- Power On the Device Press the start button to turn on the cardiocheck Duo.
- Select ECG Mode Navigate to the ecg function using the device screen or the medigo flex App.
- Place Your Fingers on the Sensors Hold the device with both hands, placing your fingers on the ECG electrodes.
- Remain Still Sit comfortably and avoid movement while the ECG is being recorded.
- Monitor the Progress The device will display a progress bar or indicate when the ECG reading is complete.
- View Results The ECG reading will be displayed on the screen and automatically saved in the medigo flex.
- Analyze & Share Review your ECG data in the app, & if needed share it with your healthcare provider for further analysis.

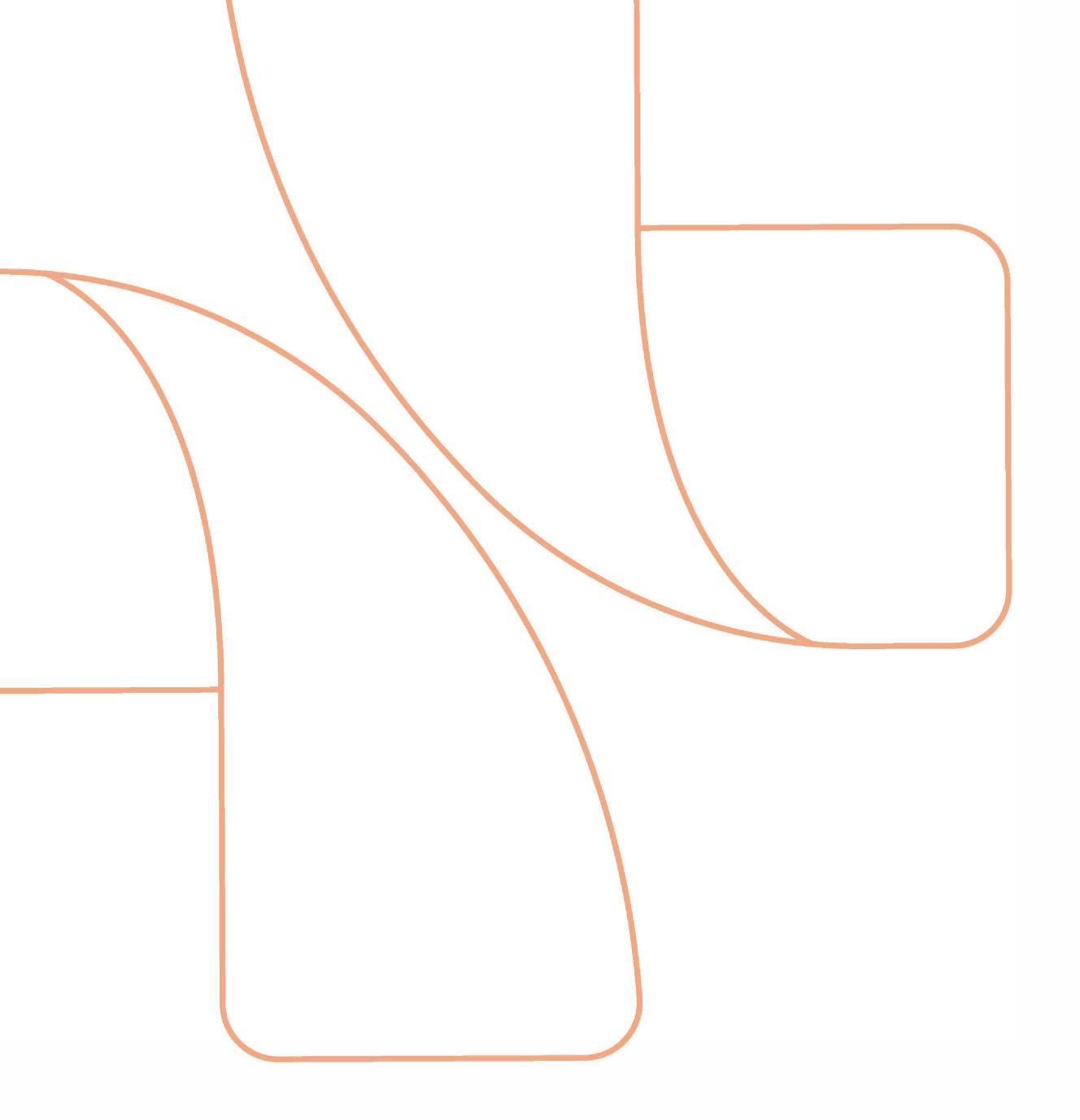

#### About MediGo Flex:

At MediGo Flex, we prioritize the security and privacy of your health information. Access to personal health records is strictly controlled and granted only to authorized users.

Our devices are meticulously created with a user-friendly design, tested against various devices, and seamlessly integrate into your daily routine.

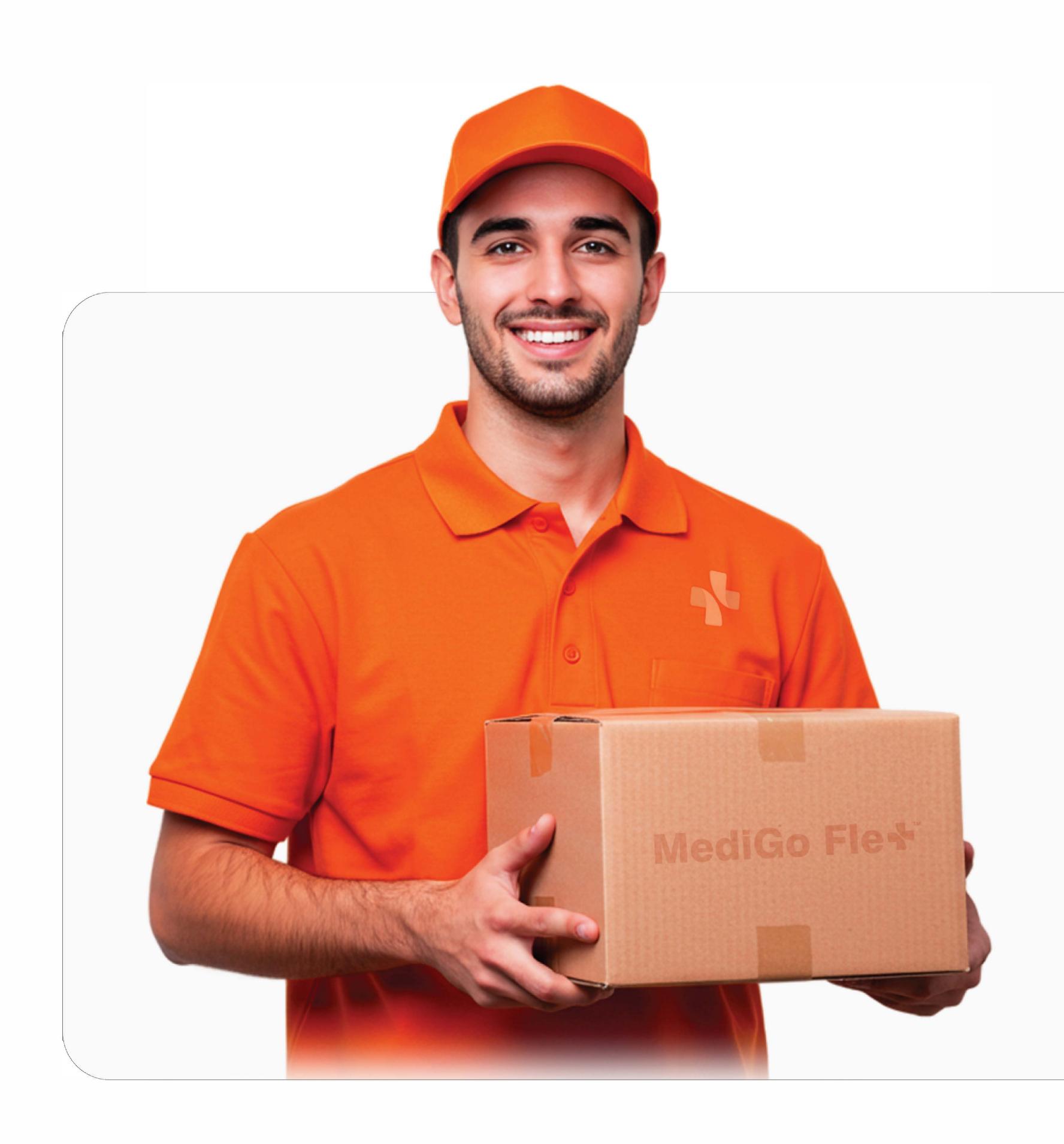



#### Order Now

### CardioCheck Duo

Experience seamless health monitoring with the CardioCheck Duo.

Visit Our Website to Place Your Order Today!

### Contact Us

+918128613883 support@medigoflex.com